

## Asepti Alcohol Wipes

Hospital Grade Disinfectant Wipes - 80xSoft Pack



### Features

- Asepti Alcohol Wipes are easy to use 75% v/v ethanol based surface wipes for cleaning and disinfecting surfaces
- Asepti Alcohol Wipes can be used on most hard surfaces except therapeutic devices and are suitable for use in hospitals, doctor and dental surgeries and other healthcare facilities
- Asepti Alcohol Wipes assist in the cleaning of surfaces and remain effective while the cloth remains moist
- Asepti Alcohol Wipes are alcohol impregnated wipes suitable to clean and disinfect most environmental hard surfaces
- Asepti Alcohol Wipes are conveniently packed in a resealable soft pack format. Each wipe is delivered fresh and moist. The cloths are no lint, no foam and are fast drying
- Asepti Alcohol Wipes are a no mess, no fuss, general purpose surface wipe for pre-cleaned surfaces
- Asepti Alcohol Wipes may be used on dry laminate and hard surfaces except therapeutic devices
- For best results apply to dry, previously cleaned surfaces in a normal wiping motion
- Asepti Alcohol Wipes will assist in the hygienic cleaning of a surface and will remain effective while the cloth remains moist
- Protective gloves are not required, however you should wash your hands after use

Product Code	Product Description	Pack Size	Quote
7400288	Asepti Alcohol Wipes	12 x 80 Soft Pk	
99001137	Plastic Holder to suit Flat Pack	ea	
99960090	Wire Holder to suit Flat Pack	ea	

### Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.

ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

## Directions For Use

---

- Open the resealable tab.
- Extract wipes from the outlet.
- Reseal using the adhesive tab before closing the plastic lid to avoid evaporation.
- For disinfecting apply to dry, previously cleaned surface in a normal wiping motion.
- Dispose of used wipes in garbage bin.
- Do not flush wipes in toilet.
- Always wash hands after use.

## Related Products

---

- [Asepti Neutral Wipes, 325 x Pail or 100 x Soft Pk](#)
- [Asepti Active Wipes, 325 x Pail or 100 x Soft Pk](#)

## Applications

---

- Theatre
- Dental
- Laboratory
- General Medical
- Patient Room
- Patient / Public Bathrooms
- Waiting Rooms
- Veterinary
- Endoscopy
- Residential Aged Care
- Life Science
- Healthcare Facilities
- Wards
- Emergency
- Day Surgeries
- Office / Meeting Rooms

## Resources

---

- SAFETY DATA SHEET (SDS)
- Video
- Brochure