

Asepti NewGen

Non-hazardous non-enzymatic metal safe detergent providing a clean result and staff safety

ARTG 122628



Features

- The non-hazardous classification and non-caustic composition of Asepti NewGen together with its effective cleaning performance make it an excellent choice for the ward washer disinfectors.
- Asepti NewGen offers a combined objective of a clean result and staff safety.
- Asepti NewGen is a mild alkaline detergent used for manual and machine cleaning of surgical, laboratory and dental instruments.
- Asepti NewGen is soft metal safe and is suitable for cleaning medical utensils such as surgical steels, plastic, elastomers, anodized and non-anodized aluminium, titanium, anaesthetic masks, circuits and connectors.
- Asepti NewGen is suitable for use in utensil, ward, pantry, batch, ultrasonic, index, cart and trolley medical washer disinfectors.
- Asepti NewGen complies with AS/NZS 5369:2023 and AS/NZS 4815:2014

Product Code	Product Description	Pack Size	Quote
7100737, 7105473	Asepti NewGen, 5 L	2 Pk	
7101742	Asepti NewGen, 110 L	ea	

Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.

ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

Directions

Manual Cleaning:

- Recommended dilution is 2 – 8 mL per Litre.

Automated applications:

- In utensil and ward washers the recommended dilution is 5 mL per Litre.

Batch Washers, Index Washers, Cage Washers and Trolley Washer Disinfectors:

- In Batch, Index, Cage and Trolley washers the recommended dilution rates are:
 - 3 – 8 mL per Litre for medical instruments.
 - 1 – 3 mL per Litre for cage and trolley washers

Asepti NewGen is recommended to be used in conjunction with Asepti Rinse Clear or Asepti Citric Rinse in the final high temperature disinfection stage of non-anaesthetic cycles.

Relevant Product

- [Belimed WD290IQ](#)
- [Belimed WD290](#)
- [Belimed WD200](#)
- [Belimed WD390](#)
- [Belimed WD750](#)
- [Belimed CS750](#)
- [Smeg WD7015](#)
- [Smeg WD7010](#)
- [Smeg WD6010](#)
- [Smeg WD1160](#)
- [Meiko TopClean 60](#)

Applications

- Infection Control
- Healthcare Facilities
- Central Sterile Departments
- Residential Aged Care
- Life Science

Resources

- safety data sheet (sds)
- Video
- Brochure